


Performance Setting Guide

The valves perform within a tolerance range of the average pressure as shown here regardless of gravitational orientation:

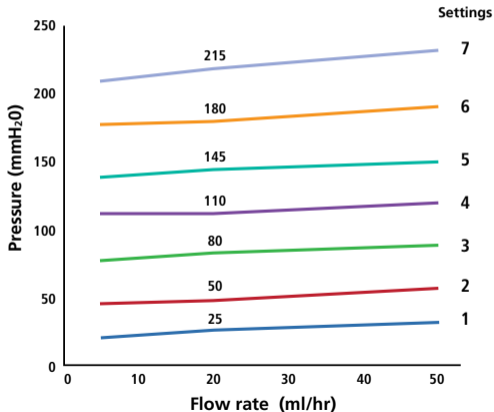

Settings 1, 2, 3 ±20 mmH₂O

Setting 4 ±25 mmH₂O

Settings 5,6,7 ±35 mmH₂O

Setting 8 is intended to limit flow through the valve and has a minimum pressure of 400 mmH₂O.

CODMAN CERTAS[®] Plus Programmable Valves were made available after March 2015

CODMAN NEURO

 **DePuy Synthes**
COMPANIES OF 

X-Ray Performance Setting Guide

Codman
CERTAS[®] PLUS
Programmable Valve

- Right Hand Side (RHS) X-Ray Marker
- Setting Indicator

CODMAN NEURO

DePuySynthes *People inspired™*
COMPANIES OF Johnson & Johnson

Codman & Shurtleff, Inc.

325 Paramount Drive
Raynham, MA 02767

For Customer Service in the USA
call: 1-800-255-2500

www.depuySynthes.com

